USP 797 & 800 Compliance Specialists

Everything you need to achieve and maintain a compliant cleanroom environment

Chemo/Hazardous Drug Clean



For Daily Removal of Hazardous Drug Surface Contamination in Preparation and Administration Areas

Hazardous Drug Clean (HDCleanTM) is an easy two step wiping procedure using towelettes that effectively removes hazardous drug surface contamination. HDCleanTM has been tested to remove surface contamination of docetaxel, paclitaxel, 5-flourouracil, cyclophosphamide, ifosfamide, and cisplatin.

HDCleanTM Advantage:

- Simple towelette wiping procedure
- Proven effective in laboratory tests
- Effective on a wide variety of hazardous drugs
- No residue left on surfaces after use
- No overpowering odor
- Larger towelettes offer expanded cleaning area up to 4 square feet

Recommended Use: HDClean[™] is recommended for cleaning hazardous drug storage, preparation, and administration areas before shift changes, at the end of the day, and after hazardous drug spill cleanup.

Directions for Use: HDCleanTM is a two step wiping process. Use the towelette marked "1" to thoroughly wipe one area. Wipe the same area using the towelette marked "2". Open up a second pack of towelettes and repeat wiping of the same area with towelette marked "1" and then "2". Repeat this procedure for each 2 ft x 2 ft area wiped. DO NOT REUSE THE TOWELETTES. Discard towelettes in chemotherapy waste bin.

Approved for use on the following surfaces: chemical resistant laminate, phenolic resin, epoxy resin, high-density polyethylene, 316 stainless steel, 304 stainless steel, 201 stainless steel. **Use with caution on other surfaces**.

Caution: For external use only. Towelette chemistry can be harmful to eyes. Immediately flush eyes with plenty of water for 15 minutes after contact. If irritation persists, call a physician. Use in a ventilated area. May cause skin irritation. **FLAMMABLE:** Keep away from fire or open flame.

Active Ingredients: Quaternary ammonium and isopropyl alcohol.



Phone: (407) 296.6689 Web: www.Carter-Health.com

Introduction

- Health care workers are exposed to hazardous drugs as a part of the dispensing and administration of medications
- Evaluation of pharmacy and nursing employees have documented the presence of hazardous drugs on both the surface of their work area and in their urine
- Studies have also been conducted that have detected the presence of chemotherapy on the outside of drug vials
- Surface wipe studies are recommended to be completed regularly in areas that prepare and administer hazardous drugs in order to understand the risk of exposure to employees
- ChemoGLO™ isawipekitthathasbeenusedinover 400 hospitals over the past 4 years
- Surface contamination for 5-FU, cyclophosphamide, ifosfamide, docetaxel, paclitaxel, and cisplatin are reported following use of the wipe kit
- As much as 80–90% of hospitals wiped have documented surface contamination of hazardous drugs, even when using all of the recommended best practices
- Because of the concerns associated with documented hazardous drug residue, HDClean™ was developed as a method to remove all detectable surface contamination
- HDClean[™] utilizes a two-step system with specialized cleaning solutions and towelettes to remove hazardous drug residue from surfaces

Objectives


• To evaluate the effectiveness of HDClean™ in removing 5-FU, cyclophosphamide, ifosfamide, docetaxel, paclitaxel and cisplatin

Methods

- A total of 14 separate areas were contaminated with 5-FU, cyclophosphamide, ifosfamide, docetaxel, paclitaxel and cisplatin
- Each area was 2 ft x 2 ft (4 ft2)
- Exposure of each drug on the 4 ft² site was 1,000 ng/ml
- \bullet Seven areas were cleaned with HDClean TM prior to sampling
 - Towelette #1 was used first to clean the area and then followed by towelette #2
 - Procedure was repeated a second time on each contaminated area
- ChemoGLO[™] wipe methods and liquid chromatography tandem-mass spectrometry (LC-MS/MS) assay for 5-FU, cyclophosphamide, ifosfamide, docetaxel, and paclitaxel, and inductively coupled plasma mass spectrometry (ICP-MS) assay for cisplatin were used for analysis

Results

- ChemoGLO[™] experience for surface contamination risks with cyclophosphamide
 - 3.4% of samples: detectable surface concentration > 1 ng/cm²
 - 7.7% of samples: detectable surface concentration 0.1-1 ng/cm²
 - 88.9% of samples: detectable surface concentration < 0.1 ng/cm²

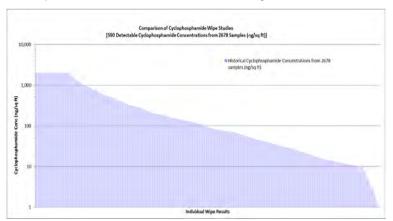

- Bench top surfaces were contaminated with 6 different chemotherapy agents (5-FU, cyclophosphamide, ifosfamide, docetaxel, paclitaxel and cisplatin) at a concentration of 1,000 ng/ml
- ullet Seven of the areas were wiped using HDClean $^{\text{TM}}$ (two-towelette system, repeated once) and 7 were not wiped
- ChemoGLO™ wipe kit was used to detect residual surface contamination of the hazardous drugs

	Total number of tests	ChemoGLO [™] measured concentrations of each drug
Samples not using HDClean™	7	900 – 1,000 ng/ml
Samples using HDClean™	7	Non-detectable


- LC-MS/MS assays for 5-FU, cyclophosphamide, ifosfamide, docetaxel, and paclitaxel had a lower limit of quantification for each drug of 10 ng/mL.
- ICP-MS assay had a lower limit of quantification for cisplatin of 1 ng/mL

Conclusions

- Surface contamination in hospitals continues to be detectable, even when all best practices are being utilized
- Health care employee risk can be evaluated with regular surface contamination monitoring
- Additional strategies and products need to be developed to minimize surface contamination of hazardous drugs and subsequent employee risk
- HDClean[™], when used according to product recommendations, demonstrated removal of all detectable surface contamination of the following chemotherapy drugs: 5-FU, cyclophosphamide, ifosfamide, docetaxel, paclitaxel, and cisplatin at exposures that have been reported in hospitals and pharmacies.

Evaluation of Hazardous Drug Clean™ (HDClean™), a two-step towelette system, in removing surface contamination of hazardous drugs

Stephen F. Eckel ^{1,5,6}, William F. McAdoo ^{1,2}, Candice A. Sherwood ^{1,2}, Sara K. O'Neal, ^{1,2} and William C. Zamboni ^{1,2},3,4,5

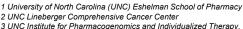

4 Carolina Center of Cancer Nanotechnology Excellence, University of North Carolina, Chapel Hill, NC 5 ChemoGLO, LLC, Chapel Hill, NC

6 University of North Carolina Hospitals





Phone: (407) 296.6689

Web: www.Carter-Health.com

Material Safety Data Sheet

May be used to comply with OSHA's Hazard Communication Standard, 29 CFR 1910 1200. Standard must be consulted for specific requirements.

U.S. Department of Labor

Occupational Safety and Health Administration (Non-Mandatory Form) Form Approved OMB No. 1218-0072

		OMB No	. 1218-0072				
IDENTITY (as Used on Label and List) HDClean 1			Note: Blank spaces are not permitted. If any item is not applicable or no information is available, the space must be marked to indicate the				
Section I		'					
Manufacturer's name ChemoGLO, LI	_C	Emergen	Emergency Telephone Number (919) 428-6759				
		Telephon	Telephone Number for Information (919) 428-6759				
96 Mountain Laurel	Date Prep	Date Prepared 11/28/2012					
Chapel Hill, NC 27517		Signature	of Preparer (op				
Section II—Hazardous Ingredier	ts/Identity Information						
Hazardous Components (Specific Chemical le	dentity, Common Name(s))		OSHA PEL	ACGIH TLV	Other Limits Recommended	% (optional)	
Octyl decyl dimethyl Ammonium ch	loride (32426.11-2)		N/A	N/A	N/A	0.025	
Dioctyl dimethyl Ammonium chlorid	· ,		N/A	N/A	N/A	0.010	
Didecyl dimethyl Ammonium chloric	de (7173-51-5)		N/A	N/A	N/A	0.015	
Alkyl (ci2, 40%; Cic, 10%) dimethyl	benzyl ammonium chloride (68	8424-85-1)	N/A	N/A	N/A	0.034	
Section III—Physical/Chemical (Chaoifia	Gravity (H20 = 1	\	I 1/4		
Boiling Point	N/A)	N/A		
Vapor Pressure (mm Hg)	N/A	Melting F			N/A		
Vapor Density (AIR = 1)	Unknown	Evapora	tion Rate (Butyl <i>i</i>	tyl Acetate = 1) Same as \		e as Water	
Solubility in Water Complete							
Appearance and Odor Foil packaged	moist towelette, pleasant odd	or					
Section IV—Fire and Explosion	Hazard Data						
Flash Point (Method Used) N/A		Flammab	e Limits N/A	LEL N/A	UEL N/A		
Extinguishing Media Co2 – Dry Chem	ical – Foam – Water Spray						
Special Fire Fighting Procedures Co2 -	- Dry Chemical – Foam – Wat	er Spray					
Unusual Fire and Explosion Hazards	one Known						

(Reproduce locally) 1985

OSHA 174 Sept.



Carter-Health Disposables, LLC Phone: (407) 296.6689

Web: www.Carter-Health.com

Section V—	Reactivity Data			
Stability		Unstable		Conditions to Avoid N/A
		Stable	Х	
Incompatibility (I	Materials to Avoid) N/A	À		
Hazardous Deco	emposition or Byproducts	Carbon monoxide a combustion.	nd other unident	ified organic gases may occur during in complete
Hazardous		May Occur		Conditions to Avoid N/A
Polymerization		Will Not Occur	Х	
Section VI-	-Health Hazard Data			
Route(s) of Entry		Inhalation? N/A	Skin? X	Ingestion? X
Health Hazards		<u> </u>		Skin: redness, possible skin irritation.
Swallowing	ı: probable mucosa		, ,	
	•			
Carcinogenicity	None Known	NTP? N/A	IARC Mo	nographs? N/A OSHA Regulated? N/A
Signs and Syr	nptoms of Exposure No	one Known		
Medical Conditio Generally Aggra	rated by Cympaying	e Known		
	First Aid Dussedons			
Emergency and	First Aid Procedures	Eye contact: flush imme	ediately with wate	er for 15 minutes. Call a physician.
physician.				with water for 15 minutes. If irritation persists call a hese products are not available drink large quantities
Section VII-	-Precautions for Sa	fe Handling and Use		
Steps to Be Tak	en in Case Material Is Relea	sed or Spilled N/A		
Waste Disposal	Method Dispose of	used towelettes in cher	motherapy waste	e bin.
Precautions to B	e Taken in Handling and Sto	Handling: N/A.	Storage: Keep	away from heat or flame.
Other Precaution	ns Keep away fron	n children.		
	-Control Measures			
		N/A		L Ownering
Ventilation	Local ExhaustN/A			Special N/A
	Mechanical (General)	N/A	_	Other N/A
Protective Glove	s N/A		Eye F	Protection N/A
Other Protective	Clothing or Equipment	N/A		
Work/Hygienic F	ractices Wash hand	ds after using towelettes	S	



Phone: (407) 296.6689 Web: www.Carter-Health.com

Material Safety Data Sheet

May be used to comply with OSHA's Hazard Communication Standard, 29 CFR 1910 1200. Standard must be consulted for specific requirements.

U.S. Department of Labor

Occupational Safety and Health Administration (Non-Mandatory Form) Form Approved OMB No. 1218-0072

DENTITY (as Used on Label and List)						
DENTITY <i>(as Used on Label and List)</i> IDClean 2	Note: Blank spaces are not permitted. If any item is not applicable o no information is available, the space must be marked to indicate that.					
Section I		•				
Manufacturer's name ChemoGLO, LLC		Emergency Telephone Number (919) 428-6759				
ddress (Number, Street, City, State and ZIP Code)		Telephone Number for Information (919) 428-6759				
96 Mountain Laurel		Date Prepared 11/28/2012				
Chapel Hill, NC 27517		Signature of Preparer ((optional)			
Section II—Hazardous Ingredients/Identity	/ Information	•				
lazardous Components (Specific Chemical Identity, Comm	on Name(s))	OSHA PEL	ACGIH TLV	Other Lim Recommen		% (optional)
2-propanol 67-63-0	(Vacated (Vacated		TWA: 200 ppm STEL: 400 ppm	NIOSH IDLH 2	2000 ppm	50-100%
Section III—Physical/Chemical Character	etine					
Section III—Physical/Chemical Characteri		Specific Gravity (H ₂ 0	= 1)		0.78	I6 0 788
ioiling Point	stics 82.3C	Specific Gravity (H ₂ 0	= 1)		0.78	6-0.788
		Specific Gravity (H ₂ 0 Melting Point	= 1)		0.78	
ioiling Point	82.3C	, , , -	,			
oiling Point apor Pressure (mm Hg)	82.3C 33 mg Hg at 20C	Melting Point	,		-880	
apor Pressure (mm Hg) apor Density (AIR = 1)	82.3C 33 mg Hg at 20C 2.07	Melting Point Evaporation Rate (Bu	,		-880	
oiling Point (apor Pressure (mm Hg) (apor Density (AIR = 1) (olubility in Water Miscible	82.3C 33 mg Hg at 20C 2.07 owelette, alcohol odd	Melting Point Evaporation Rate (Bu	,		-880	
oiling Point apor Pressure (mm Hg) apor Density (AIR = 1) olubility in Water Miscible ppearance and Odor Foil packaged moist to	82.3C 33 mg Hg at 20C 2.07 owelette, alcohol odd	Melting Point Evaporation Rate (Bu	,	%	-880	
apor Pressure (mm Hg) apor Density (AIR = 1) apor Density in Water Miscible appearance and Odor Foil packaged moist to be consisted to be consisted and Explosion Hazard Dates	82.3C 33 mg Hg at 20C 2.07 owelette, alcohol odo	Melting Point Evaporation Rate (Bu	ityl Acetate = 1)	%	-880 1.5	
colling Point apor Pressure (mm Hg) apor Density (AIR = 1) colubility in Water Miscible ppearance and Odor Foil packaged moist to collection IV—Fire and Explosion Hazard Dallash Point (Method Used) 35C axtinguishing Media Co2 – Dry Chemical – A	82.3C 33 mg Hg at 20C 2.07 owelette, alcohol odo	Melting Point Evaporation Rate (Bu or Flammable Limits m — Dry Sand	LEL 2	%	-880 1.5	

(Reproduce locally) 1985 OSHA 174 Sept.



Phone: (407) 296.6689 Web: www.Carter-Health.com

Section V—Reactivity Data					
Stability	Unstable		Conditions to Avoid Heat, flame, sparks		
	Stable	Х			
Incompatibility (Materials to Avoid)	trong oxidizing agents,	acids, halogens,	acid anhydrides		
		, ,	,		
Hazardous Polymerization	May Occur		Conditions to Avoid N/A		
,	Will Not Occur	X			
Section VI—Health Hazard Da	ta	1	1		
Route(s) of Entry Eye	Inhalation? X	Skin? X	Ingestion? X		
Health Hazards (Acute and Chronic)	Eye: redness, possible	eye damage. S	kin: redness, possible skin irritation.		
Inhalation: possible irritation					
Carcinogenicity None Known	NTP? N/A	IARC Mo	nographs? N/A OSHA Regulated? N/A		
Signs and Symptoms of Exposure In	halation: drowsiness/diz	zziness. Ingestio	n: nausea, vomiting		
Medical Conditions Generally Aggravated	hy Evnouro				
ivietical Continuous Generally Aggravated	None Known	<u> </u>			
Emergency and First Aid Procedures	Eve contact: flush	immediately w	ith water for 15 minutes. Call a physicia	n	
Skin: if irritation develops, imme Ingestion: do not induce vomitin	diately stop using product g. Call a physician. Inh	and wash with wate alation: move away	er for 15 minutes. If irritation persists call a physician. from product to a well ventilated area.		
Section VII—Precautions for S	Safe Handling and Use				
Steps to Be Taken in Case Material Is Rele	eased or Spilled N/A				
Masta Discussed Mathad					
Waste Disposal Method Dispose o	f used towelettes in che	emotherapy waste	bin.		
Processions to Be Taken in Handling	and Staring				
	Handling: N/A	A. Storage: Stor	e in a cool, dry space. Keep away from heat or flar	ne.	
Other Precautions Keep away	from children				
1.00p away	nom omaron.				
Section VII—Control Measure	 S				
Respiratory Protection (Specify Type)	N/A				
Ventilation Local Exhaust N/A			Special N/A		
Mechanical (General)	N/A		Other Use in a well ventilated area		
Protective Gloves N/A		Eye Prot			
Other Protective Clothing or Equipment	N/A	I			
Work/Hygienic Practices Wash har	nds after using towelette	es.			